

# RapID™ Systems for Identification of:

Anaerobes  
Corynebacterium  
Enterobacteriaceae

Neisseria-Haemophilus  
Nonfermenters-Vibrio  
Staphylococci

Streptococci  
Yeast  
Urinary tract isolates



**Ease of Use:** Simple, one-step inoculation

**Time Savings:** Accurate, easy-to-read, color reactions in 4 hours or less.

**Labor Savings:** Only a few minutes of hands-on time for set-up and read-out.

**Accurate:** Extensive electronic code compendium (ERIC®) for easy look up of RapID™ code interpretation.

**Extensive Database:** Over 400 medically important organisms identified by RapID™ Systems.

See the RapID™ Quick Reference Guide on the reverse

992-120

© 2009 Thermo Fisher Scientific Inc. All Rights Reserved. All trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

800-255-6730 General  
913-895-4128 International Fax

12076 Santa Fe Drive  
Lenexa, KS 66215

**remel**  
Part of Thermo Fisher Scientific

# RapID™ Systems and Accessory Products Quick Reference Guide

RapID™ Systems	Organisms Identified	Panel 20/pk	Inoculation Fluid 20/pk	Incubation Temperature (aerobic)	Incubation Time	Spot Indole Reagent R8309002	Nitrate A Reagent R8309003	Nitrate B Reagent R8309004	Oxidase Reagent R21540	McFarland Standard	Color Guide	QC Organisms
RapID™ ANA II	Anaerobes	R8311002	R8325102	35°-37° C	4 hours	√	-	-	-	#3 - R20413	992-143	R4653050
RapID™ CB PLUS	<i>Corynebacterium</i> , coryneform bacilli	R8311008	R8325106	35°-37° C	4 hours	-	√	√	-	#4 - R20414	992-149	R4653048
RapID™ NH	<i>Neisseria</i> , <i>Haemophilus</i> , <i>Moraxella</i>	R8311001	R8325102	35°-37° C	4 hours	√	√	√	√	#3 - R20413	992-142	R4653051
RapID™ NF PLUS	Oxidase +, Gram - Nonfermenters/ <i>Vibrio</i>	R8311005	R8325102	35°-37° C	4 hours	√	√	-	√	#1 - R20411	992-146	R4653054
RapID™ ONE	Oxidase -, Gram - bacilli ( <i>Enterobacteriaceae</i> )	R8311006	R8325106	35°-37° C	4 hours	√	-	--	√	#2 - R20412	992-147	R4653056
RapID™ STAPH PLUS	Staphylococci (40)	R8311009	R8325106	35°-37° C	4 hours	-	√	√	-	#3 - R20413	992-150	R4653047
RapID™ SS/u	Urinary tract pathogens	R8311004	R8325102	35°-37° C	2 hours	√	-	-	-	#1 - R20411	992-145	R4653053
RapID™ STR	Streptococci (over 30 taxa)	R8311003	R8325102	35°-37° C	4 hours	-	-	-	-	#1 - R20411	992-144	R4653052
RapID™ YEAST PLUS	Yeast (over 40 taxa)	R8311007	R8325106	30° C	4 hours	-	-	-	-	Wickerham Card	992-148	R4653060

All Color Guides are available at no charge and are available from either Technical Support or your local Technical Sales Representative. ERIC® Code Compendium, product number R8323600 is a CD ROM Windows®-based software program for determination of organisms identified using the RapID™ Systems.